



Safe, highly effective technology in a simple solution

- Includes the naturally occurring molecule hypochlorous acid
- Rapidly kills 99.999% of *MRSA*, *VRE*, *Acinetobacter baumannii*, *E.Coli*, and other infectious pathogens in solution in just 15 seconds in laboratory testing with no known microbial resistance*
- Demonstrated to be extremely safe, gentle, and hypoallergenic: just apply directly to wounds, even around eyes, mouth, and nose
- Rigorously safety tested: non-cytotoxic, non-irritating, non-sensitizing, non-mutagenic, no oral toxicity
- No antibiotic resistance
- No clinical contraindications for use

If you have any treatment-related questions, contact your clinician at the number provided. For product questions, please contact PuriCore at 1.866.918.2743.

* Contact PuriCore for a full list of antimicrobial test data.

vashe

wound therapy

Vashe Wound Therapy Solution is intended for cleansing, irrigating, moistening, and debriding acute and chronic dermal lesions such as Stage I-IV pressure ulcers, stasis ulcers, diabetic ulcers, post-surgical wounds, first- and second-degree burns, abrasions, and minor irritations of the skin in addition to moistening and lubricating absorbent wound dressings.

Vashe Wound Therapy Solution is a safe, biocompatible solution that includes hypochlorous acid (HOCl). In the Vashe Wound Therapy Solution, which passes USP 51 antimicrobial preservative test requirements, HOCl functions as a preservative and prevents the growth in solution of infectious pathogens.

Vashe Wound Therapy is available through your local authorized medical products distributor.

© 2010 PuriCore, Inc. All rights reserved. PuriCore and Vashe registered trademarks of PuriCore, Inc. All other trademarks are property of their respective owners.

P/N 0001-0286 Rev. A

PuriCore
Pure Science. Pure Life.

RxOnly

508 Lapp Road
Malvern, PA 19355
www.puricore.com
1.866.918.2743

Authorized Distributor:

vashe[®]

wound therapy



A Safe, Natural, Effective Dermal Defense Solution for

- Wound Cleansing
- Debridement Irrigation
- Dressing Management

Patient Use Guide



Building on the body's natural defense

The body knows a lot about wound healing. That's why we've developed a technology that mimics the body's production of a small molecule called hypochlorous, which is generated by white blood cells to fight harmful bacteria, viruses, and other organisms. PuriCore uses a simple combination of all-natural water and salt with electricity to produce the same powerful solution found in the body's immune system.

Vashe Wound Therapy is a pure, skin-friendly, hypoallergenic solution that is applied directly to wounds to cleanse them and remove dead tissue. It won't sting or burn. Vashe is a clinically proven solution that is used by wound care centers and hospitals throughout the county.

Vashe Wound Therapy an effective, all-natural way to cleanse wounds and minor abrasions of the skin.

A simple way to clean your wounds

At each dressing change:

1	Remove any old dressing from the wound.
2	Irrigate the wound and the surrounding intact skin with Vashe Wound Therapy Solution to remove debris and excessive wound drainage.
3	Refresh the treated area with additional Vashe Wound Therapy Solution. In deep wounds, allow Vashe Wound Therapy Solution to pool in the wound as instructed by your clinician.
4	Saturate a gauze pad or a new wound dressing with the Vashe Wound Therapy Solution, then apply it to the wound bed for the recommended soak time (see clinician's instructions on the adjacent page).
5	Keep the dressing moist by applying additional Vashe Wound Therapy Solution as needed during the soak time (every 3-5 minutes).



Step 4: Saturate a gauze pad.



Step 5: Keep the dressing moist during the soak time.

Clinician's instructions for using Vashe Wound Therapy

At each dressing change:

Follow the five steps on the adjacent page or as instructed below.

In **Step 2**, irrigate the wound with approximately _____ ml of Vashe Wound Therapy Solution.

In **Step 4**, saturate a gauze pad or new wound dressing with approximately _____ ml of Vashe Wound Therapy Solution, then apply it to the wound bed for _____ minutes.

Additional instructions:

Clinician's Signature

Clinician's Phone Number